

With focused research and testing, 7E has developed a Cotton Tip probe that is compatible with all three of our MyoLift™ devices. With a clean and sleek design, the double-ended probes deliver a precise and defined point of contact on the skin.

Both the MyoLift™ Classic Applicator Tip and the MyoLift™ Cotton Tip probes have endless benefits both separately and combined. The probes communicate with each other to deliver a precise current to the face and neck via Replenish Hydrating Spritzer and/or Conductive Tx gel. While using the pinch and hold technique, you're assured that the probe communication is taking place.

The movements for the Cotton Tip probes remain the same as the Classic Applicator probes and professionals find it is ideal to have both in the treatment room.



7e Wellness Corporation,
5858 Dryden Place,
#201 Carlsbad, CA 92008
Tel: 888-412-3160, Fax: 858-366-9831
Email: info@7ewellness.com
Website: www.7ewellness.com

COTTON TIP PROBES

MyoLift™ Cotton Tip Probes

- Constructed with Brass Alloy material (no nickel).
- Provides a smooth slide over the skin.
- Sensation of a stronger current, therefore less intensity is needed.



Benefits:

- Fits any brand of cotton tips with a paper stem.
- Less conductive gel is required.
- Pinch-and-Hold technique capabilities.

Basic Protocol:

- Plug your Cotton Tip Lead Wire into your MyoLift™ device.
- Cut cotton tip swabs to measure ¼ inch.
- Soak cotton tip swabs in Replenish Hydrating Spritzer to saturate.
- Insert tips ends into each probe socket.
- Prepare the skin prior to your MyoLift™ treatment.
- Cotton tips must stay moist in order to maintain conductivity.
- If necessary you may dip cotton tips into Replenish Hydrating Spritzer throughout your treatment to maintain moisture.
- Use your Cotton Tip probes with the same movements as your Classic Applicator Probes.
- Wipe the probe applicators thoroughly with a damp cloth to remove any residue. DO NOT IMMERSER THE METAL PORTION OF THE PROBE APPLICATORS INTO ANY LIQUID.

If you are a MyoLift™ owner, you can refer to the Quick Reference Guide in your customer support portal for more guidance.

**For more information please visit us at:
www.7ewellness.com**